


BITE-BUMPER™

opens deep bite without trauma

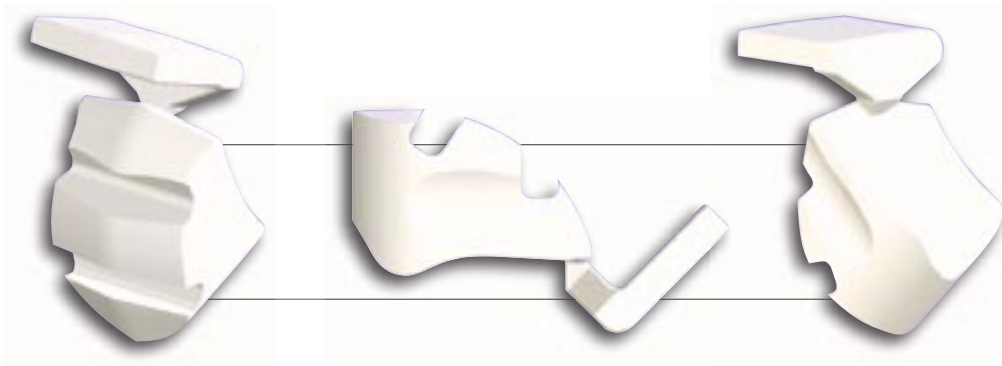

Bite-Bumpers work in open and deep bite cases by allowing the anteriors to ride up and out of occlusion. The Bite-Bumpers eliminate bond failures of brackets and are especially valuable with ceramic or self-ligating brackets.


The unique fixed "placement-jig" guarantees the correct height on the lingual of the tooth and makes sure the two Bite-Bumpers are 100 % parallel.

The Bite-Bumpers are made of a composite polymer that is more comfortable, aesthetic and will increase biocompatibility in nickel sensitive patients.

Bite-Bumpers are bonded in the same manner as metal or ceramic brackets. The Bite-Bumpers have a slotted mechanical bonding base.

The placement-jig can easily be removed by hand or by a cutting instrument after bonding - see illustration.


INSTRUCTIONS

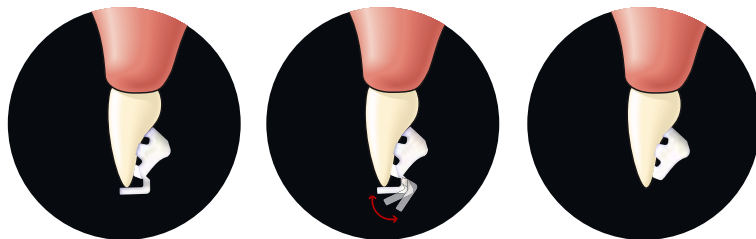

1. Put the placement-jig against the incisal edge

2. Remove the jig

3. Bite-Bumper ready

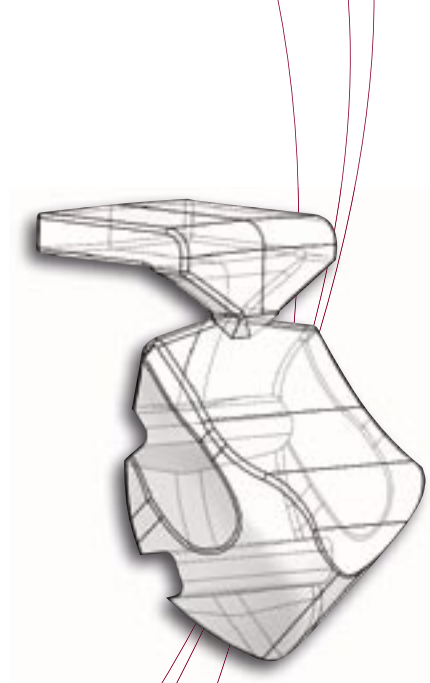

Orthodontic | Service | Europe

ATS ORTHO GmbH & Co. KG
An der Feldriede 2 | 49205 Hasbergen | Germany

☎ +49 (0) 54 05 / 92 04-0
☎ +49 (0) 54 05 / 92 04-15
@ service@ats-ortho.de

www.ats-ortho.de

Bite-Bumper™

ption	Catalogue No.
Bumper of 10)	BB-00-01